



Herts Careline
Because independence
is important

Step 2 - Vivago Care Watch - How to Set The Date

Herts Careline:
+44 (0) 300 999 2999

Living Safely:
+44 (0) 20 8798 3623
Livingsafely.com
info@livingsafely.com



Press and hold **light** and **mode** buttons together
(Skip this step if you are still in the menu after setting the time)



Press the **alarm** button to select **year** and use **light** button to select correct year



Press the **mode** button to switch to **decade** and use **light** button to select correct decade



Press **alarm** button to reach "**mo.**" (for "month") and use **light** button to select correct month



Press **alarm** button to reach **Day** and use **light** button to select correct day



Press **alarm** button to reach **View** and use **light** button to change date display
(Month:Day - Day/Night - Day of the week - Off completely)



Press and hold **mode** button when **date** is correct to return to normal function